

AB

Superior Implant Technology

D4

IMPRESSION
TAKING WITH
A CLICK



D4



IMPRESSION TAKING WITH A CLICK

D4 – The new kit: Plastic Snap Transfer with Abutment enables quick and simple impression taking as the closed tray technique, while obtaining maximum precision of the open tray technique.

The kit is designed for multiple uses and available in 4 heights (1-4mm).

It consists of the following parts:

- PK-D2 Transfer
- PK-P3 Abutment

User manual:

- Remove the healing cap from the implant
- Screw the abutment (PK-P3) into the implant
- “Click” the transfer (PK-D2) onto the abutment
- Take impression using the closed tray technique
- Remove the impression tray from the patient’s mouth, making sure that the transfer stays inside the impression material
- Remove the abutment from the implant
- Replace the healing cap into the implant
- Screw the analog (D1) to the abutment and place this unit (analog-abutment) into the transfer, inside the impression
- Send to the lab the impression tray with the above mentioned devices

www.ab-dent.com