

# INSTRUCTIONS

For using prosthetic accessories with the flat connection platform

## P12 Flat Connection

\*Under Patent

AB™




An innovative  
solution for  
**TILTED**  
implants

\*P12 Flat Connection Abutment fits the AB Dental double platform implants to the utmost precision

# P12

Flat Connection

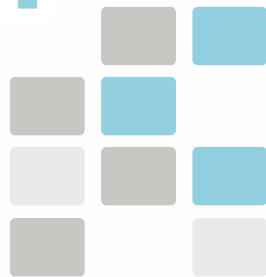


AB Dental introduces a simple and convenient solution that enables restoration on non-parallel implants by cancelling the hex and thereby cancelling the obstruction of the insertion path.

## INSTRUCTIONS FOR USING THE FLAT CONNECTION ABUTMENT


### Temporary Restoration

When performing a temporary restoration, a temporary flat connection abutment P12-T or P12-TL should be mounted on the implant and attached with a hardened composite adhesive or acryl to the temporary bridge that is prepared in advance. The structure may be shortened as required.

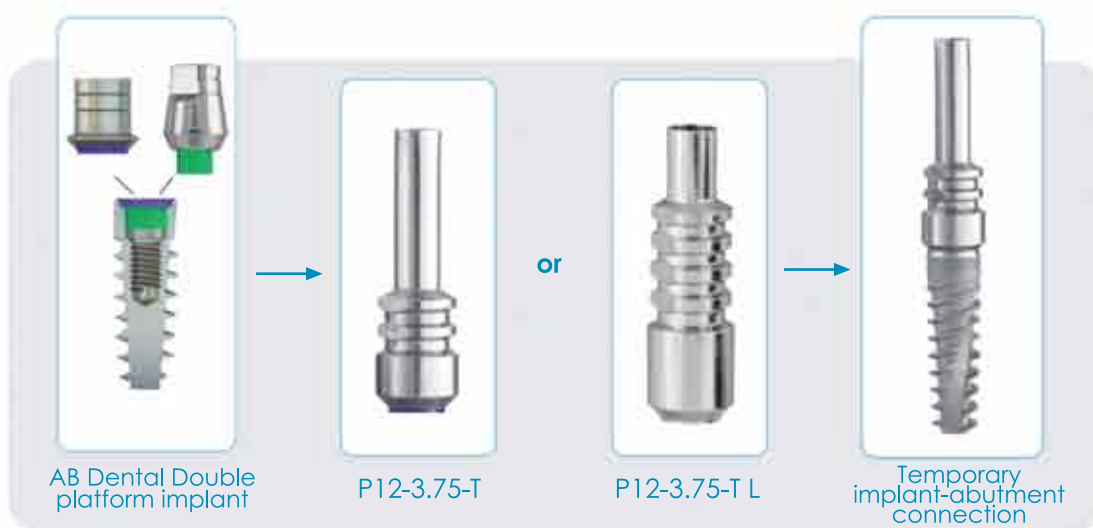

### Impression Taking

Attaching the flat impression transfer - D2-P12 - to the double platform implant for impression taking.

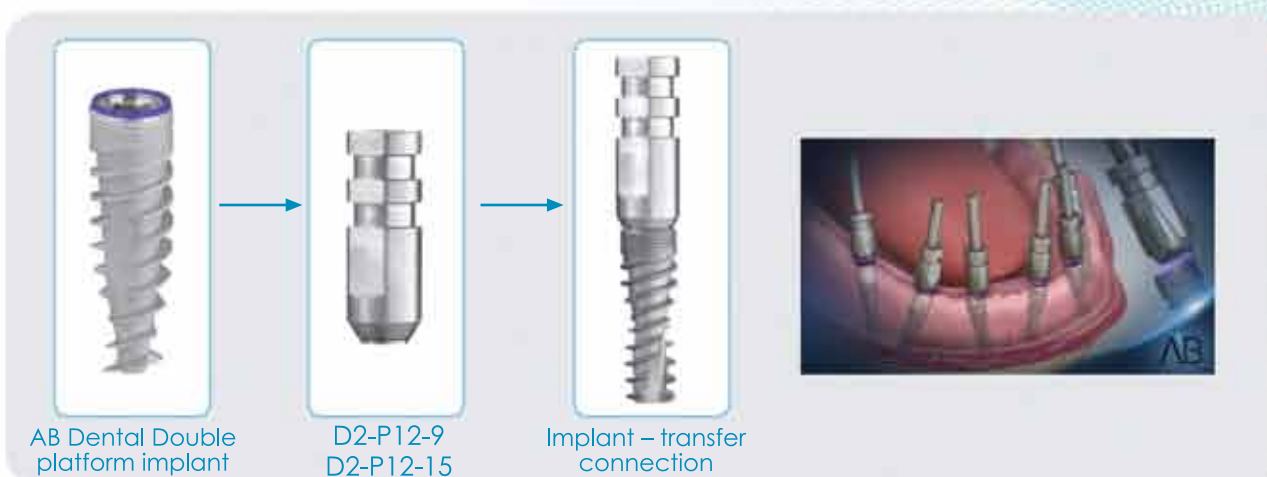



## Permanent Restoration

Assemble the permanent flat connection abutment P12-b on the analog with the laboratory model and cast the bridge with the use of a plastic sleeve P12-p.



Plastic sleeve connection

Cement the casting to the titanium abutment.



The casting is cemented to the titanium base.



Assemble the restoration on the implants using a torque of 30 N/cm<sup>2</sup> .



The restoration on a laboratory model



Restoration



## Adaptor for extension

Mount the adaptor - P12-C 3.75 - on the implant in accordance with the height of the gums with a 2.4 mm ratchet torque wrench using a torque of 30 N/cm<sup>2</sup>.



AB Dental Devices  
info@ab-dent.com  
www.ab-dent.com